## PUSH CARE KNEE BRACE

### INTRODUCTION

Push care products provide solutions when your freedom of movement is restricted by an injury or disorder. Approved by physicians and scientists.

Your new knee brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care knee brace helps to relief pain and creates a sense of greater stability by means of adjustable pressure.

Push care products are innovative and are manufactured for you with great care.

This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care knee brace.

If you have any questions, please consult your reseller or send an e-mail to [info@push-brace.com](mailto:info@push-brace.com)

### PRODUCT CHARACTERISTICS

Your Push care knee brace provides compression around the knee joint, improving the sense of stability of the joint. This compression can be adjusted as needed.

A proper anatomic fit contribute to the brace remaining in position during walking.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push care knee brace is easy to apply and can be washed without any problem.

### INDICATIONS

- Capsular inflammation
- Meniscus complaints
- After meniscus operations
- Arthrosis without hydrops
- Rheumatoid arthritis without synovitis
- Patellofemoral pain syndrome
- Functional instability without hamstring instability

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or complaints deteriorate, it is advisable to consult your doctor or therapist.


### DETERMINING SIZE / FITTING

The knee brace is available in 5 sizes. Place the thigh at an angle of 90° in relation to the lower leg. Then measure the circumference around the narrowest part below the knee joint. The table on the packaging shows the correct brace size for your knee.

A Slide the brace over the knee joint. To do this, hold the brace on both sides and pull it up the leg. Ensure that the pad is positioned next to the lower edge of the patella.

B Close the Velcro band over the thigh and apply sufficient pressure.

### WASHING INSTRUCTIONS

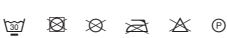Before washing, close the Velcro to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is a registered trademark of NEA International bv.

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